



Resetting a DTT tablet


1 Enter settings

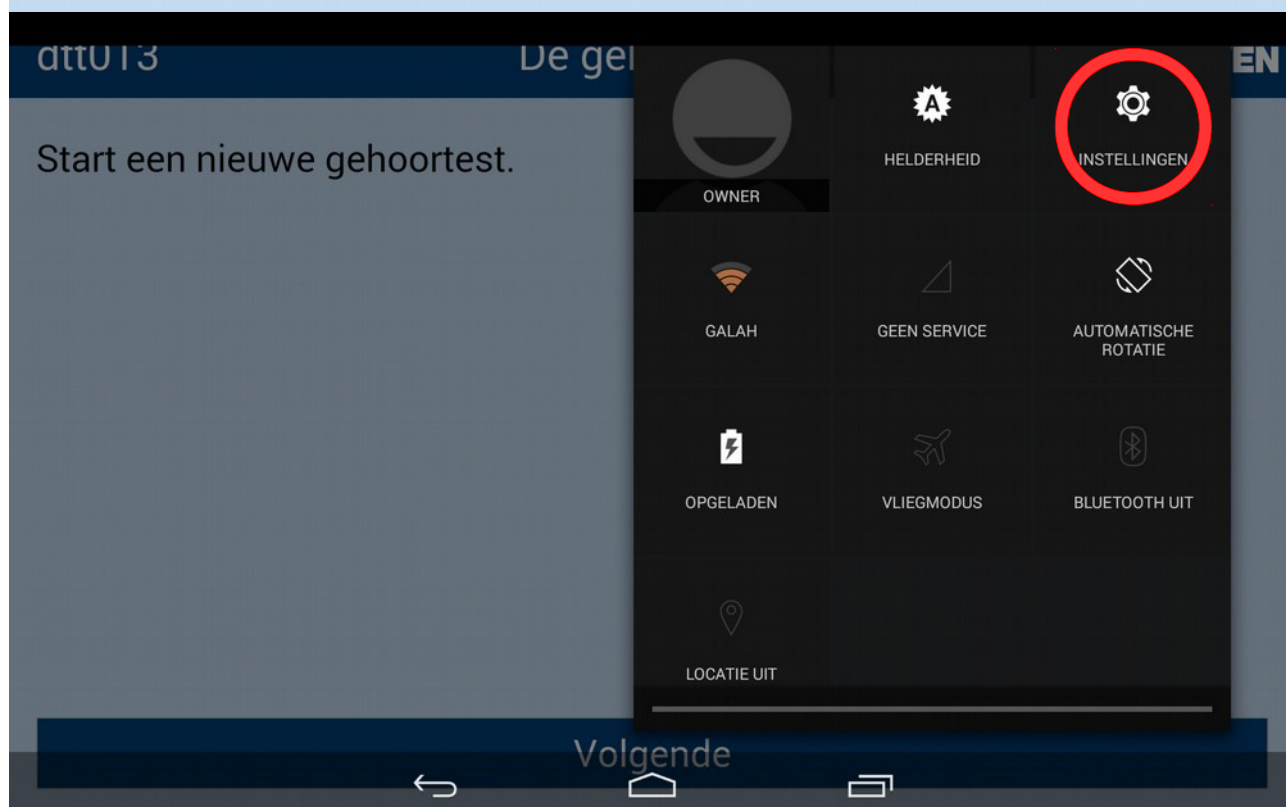


2 Manage apps

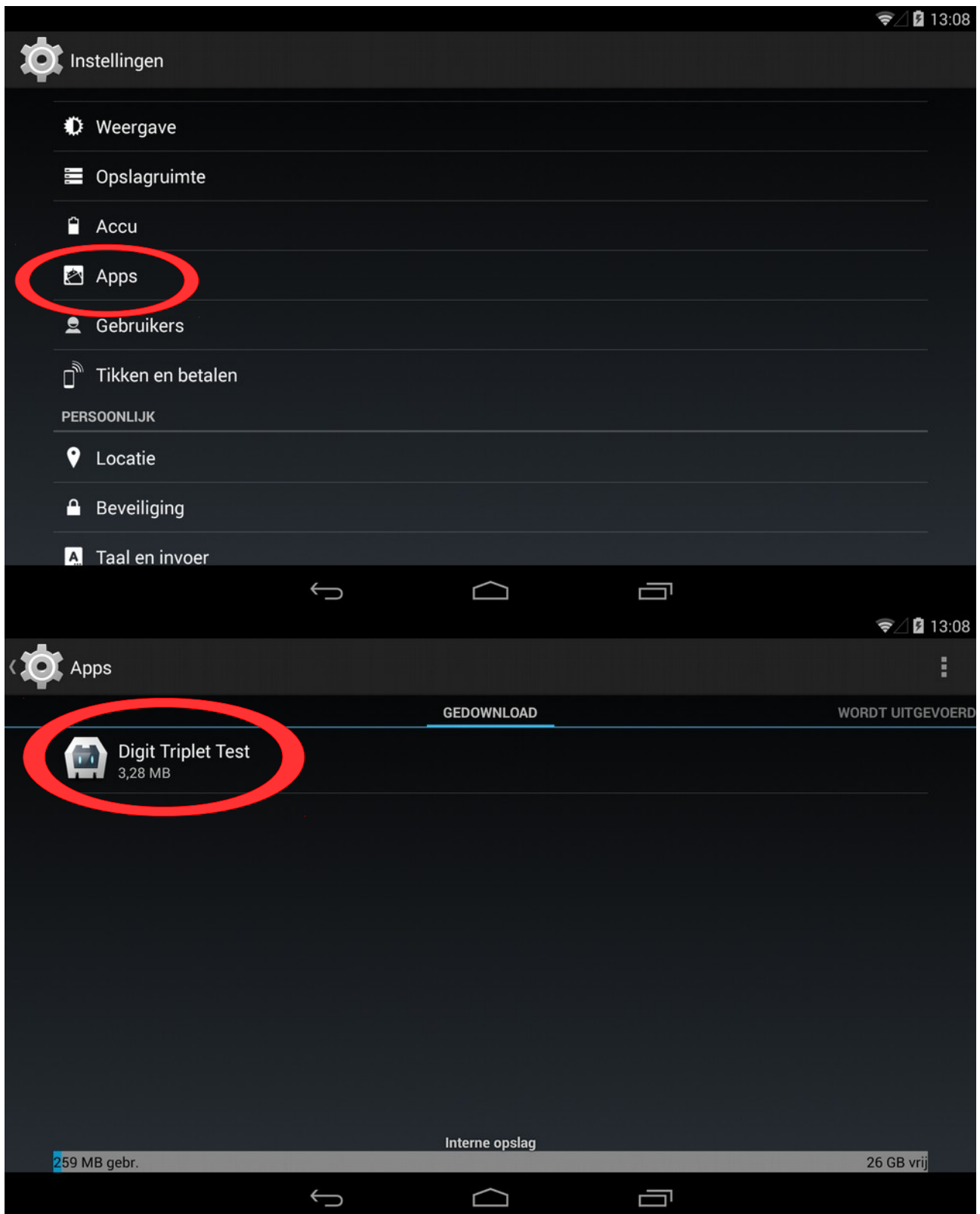

3 Remove settings

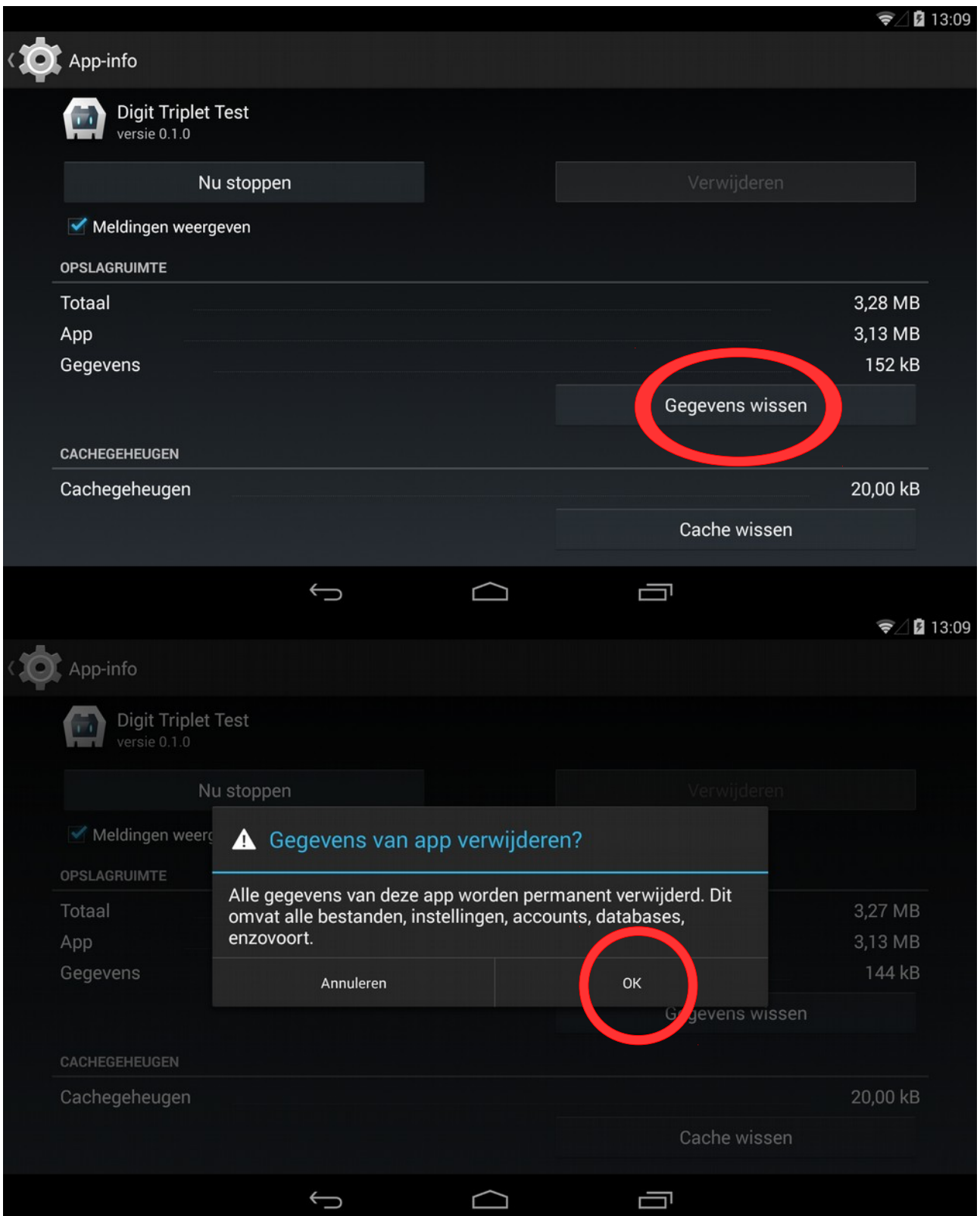

4 Stop the app

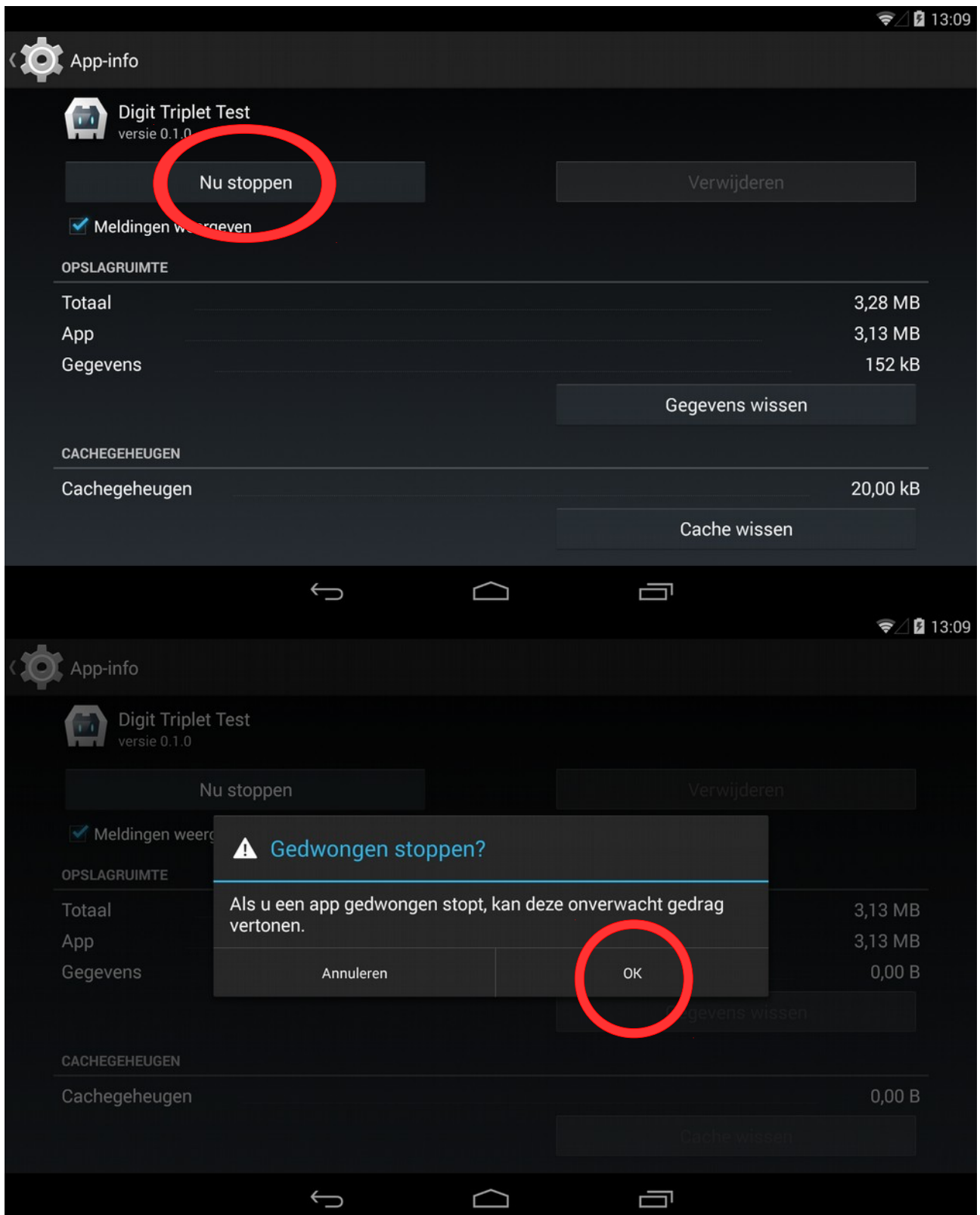

5 Restart

The image shows two parts of an Android phone screen. The top part is the 'App-info' screen for the application 'Digit Triplet Test' (version 0.1.0). It features buttons for 'Nu stoppen' and 'Verwijderen', a checked option for 'Meldingen weergeven', and storage information under 'OPSLAGRUIMTE' and 'CACHEGEHEUGEN'. The bottom part of the screen shows the app's main interface with a blue header containing 'dtt013', 'De gehoortest', and 'KU LEUVEN'. The main content area has the text 'Start een nieuwe gehoortest.' and a blue button at the bottom labeled 'Volgende'. A red circle highlights the home button in the Android navigation bar between the two screenshots.

App-info

Digit Triplet Test
versie 0.1.0

Nu stoppen

Verwijderen

Meldingen weergeven

OPSLAGRUIMTE

Totaal	3,28 MB
App	3,13 MB
Gegevens	152 kB

Gegevens wissen

CACHEGEHEUGEN

Cachegeheugen	20,00 kB
---------------	----------

Cache wissen

dtt013 De gehoortest KU LEUVEN

Start een nieuwe gehoortest.

Volgende